


**Handle for lateral flexion;
Current lock position
Release by turning clockwise
Make sure table is raised enough to turn handle
(see next picture)**

**Handle for unlocking flexion;
Current locked position
Release by turning counter-clockwise
(see next picture)**

Foot pedal for changing tension
• Right side increase
• Left side decrease

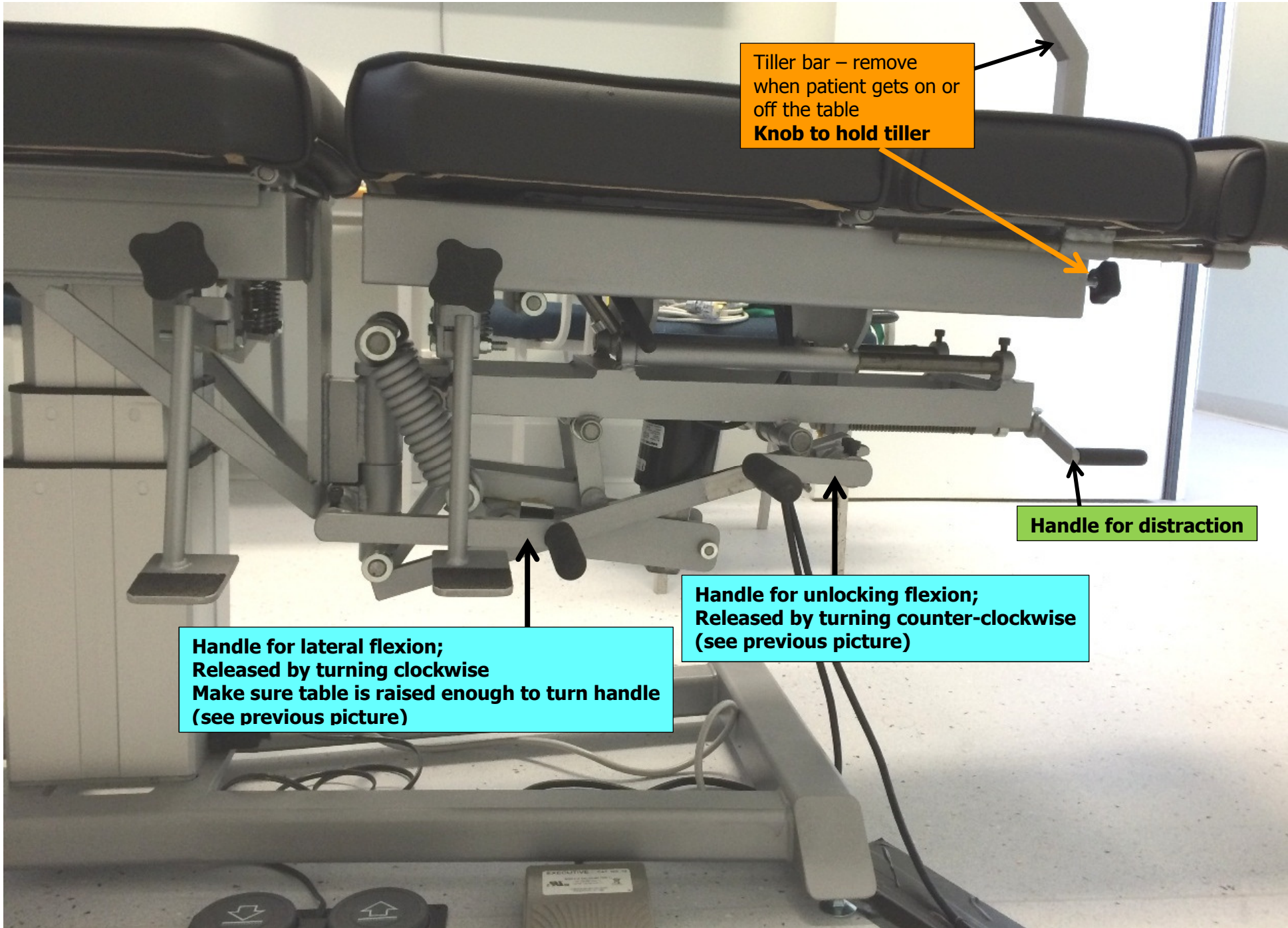

Tiller bar – remove when patient gets on or off the table
Knob to hold tiller

Handle for distraction

**Handle for unlocking flexion;
Released by turning counter-clockwise
(see previous picture)**

**Handle for lateral flexion;
Released by turning clockwise
Make sure table is raised enough to turn handle
(see previous picture)**