

Omni Wall Adjusting Board, Mounting

Many thanks for purchasing the Omni Wall Adjusting Board

To mount this item, you will need to ensure a solid fitting to your wall. With the weight of a patient and a practitioner pushing into the wall and board, will be a substantial amount of pressure. You need to ensure wherever wall you intend to mount the Omni Wall Adjusting Board can withstand this pressure.

On brick wall or a core filled brick or solid brick will require a Dyna-bolt mounting. Timber walls will you locate the vertical timber Studs to ensure you have a solid mounting.

Please refer to Figure 1 to guide you on the parts in the Omni Wall Mounting Kit.



Figure 1

Please refer to Figure 2 to guide you how the Wall Mounting Board and the Adjusting board fit together.



Figure 2